

Double Connector



Overview

The pushchair connectors comprise of three bars fixed securely with metal clamps onto the frame of each pushchair.

The Connector Bars can be detached from the clamps with hand tightened nuts leaving the clamps on the frame. If detached from one side then the Connector bars can be folded against the side of the other pushchair for convenience.

The pushchairs can be folded (individually) without removing the clamps when they are fitted at the Positions indicated.



Positioning

Note the precise positions of the Clamps A & B are less important than that they be the same on both pushchairs. Clamp C does need to be in the correct position or it could impede folding of the pushchair.

Erect the pushchair ensuring that the Red Safety Clips are engaged.

A. Handle - 3cm up from Sun Hood bracket

B. Seat Front - 10cm down from metal hinge

C. Seat Support - 16cm down from Seat Bracket, this is immediately above the frame bend

Fitting the clamps

Remove the Connector Bar from its Clamps. Refit the washer and nut on the bolt so nothing gets lost while fitting the Clamps.

1. Measure and mark the location on the frame

2. Disassemble the Clamp by undoing the screws with the allen key provided, then re-apply the two halves (including plastic inserts) with the joint in the centre of the front face of the frame.

3. Refit the screws but do not fully tighten

Repeat 1-3 for each Clamp

Test fold the pushchair to ensure none of the clamps impedes the folding. Note especially Clamp C, if this is too high it will stop the seat bracket sliding down the frame.

Repeat 1-3 on the second pushchair

4. Fit Connector Bars. Note the spring washer is located between the Clamp and the Connector Bar

5. Adjust Clamp positions as required.

6. Tighten screws to fully close the Clamps

NOTE. If you prefer you can fit the complete Connector Bar & Clamps but the screws may be a little more fiddly unless a second person assists you.

The Connector bars can be detached and the pushchairs separated, leaving the clamps on each pushchair.

Tips

1. The Clamp may be fiddly to re-attach to begin with but once the screws have started they can be tightened to fully closed compressing the plastic liner.
2. The large Connector Bar Bolts include a washer, with the bolt tightened the Connector Bar is rigid connecting the two pushchairs firmly. If the washer is removed and the bolt tightened then the joints can rotate connecting the two pushchairs more flexibly.

Operation

1. Attach both ends of the Connector Bar to clamps using the knurled knob nuts and bolts, to connect the two pushchairs.
2. IMPORTANT. Hold one handle from each pushchair when using the pushchairs. This ensures that the operator maintains safe control of both pushchairs if the Connector Bars become detached.

For use on reasonable level ground where the movement of the pushchairs is not hugely different.

When parked all four brakes must be applied.

DO NOT dismount a kerb with Connected Pushchairs instead cross where the kerb is lowered

DO NOT use on steps with the children in the pushchairs.

CHECK the fixings before each use and remedy any part which is loose.

MAINTAIN awareness of the pushchairs and the Connector Bars when in use to identify any loose parts

WARNING

Two children of 25kg each, a fairly standard 6 year old, is equal to the maximum 50kg weight for a single pushchair. If the combined weight of the two children exceeds 50kg then we recommend this is only used on flat level surfaces to ensure the carer is able to maintain safe control. Stability will be affected by the behaviour of the children, if one child has challenging behaviour they may now represent a danger to the other child.

The Connector Bar components are stronger than the pushchair frame care must be taken in operation to ensure that the weight from one pushchair does not transfer to the other pushchair and exceed its capacity.

No liability is accepted for any consequent damage to a pushchair frame resulting from use of the Double Connectors. It is almost certain that any damage would not be covered by the manufacturer's product warranty.

Parts List

- 3 x Connector Bars assembled complete
- 3 x Allen Keys
- 3 x Spare Spring washers
- 4 x Spare Clamp Screws