

RAIN COVER TO FIT

# Excel Elise NEO

## SPECIAL NEEDS PUSHCHAIR

**IMPORTANT.** This Rain Cover **DOES NOT** fit the Excel Elise, Mobiquip XL or Major Elite which have square frames

## FITTING INSTRUCTIONS

**START AT THE TOP** - PLACE THE BROAD ELASTIC AROUND BOTH PUSHCHAIR HANDLES

**NEXT THE SIDES** - PRESS THE CLIPS ONTO THE ROUND FRAME

Note. In cold weather the plastic clips may be less flexible and stiff to fit over the frame, warm first before installation.

**LASTLY THE BASE** - OPEN THE RAIN COVER AND PULL DOWN TO THE FRONT OF THE PUSHCHAIR THEN FIT THE ELASTICATED HOOKS AROUND THE FRAME BELOW THE FOOT REST

**NOTE WHEN FOLDING THE PUSHCHAIR** you can leave the Rain Cover on, it does not have to be removed. Simply detach the elasticated hooks by the footrest and close the Rain Cover back to the seat back before folding the pushchair as normal. If you do leave the Rain Cover on then it may prevent you from engaging the clip which holds the pushchair together when folded.

## SAFETY ADVICE

Do not deploy this cover when indoors or in a hot atmosphere. Keep it folded back and secured using the elasticated hooks.

Never leave this product exposed to extreme heat e.g. on the parcel shelf of a vehicle on sunny day, by a radiator or near an open fire.

FOR THE SAFETY OF YOUR CHILD, PLEASE DESTROY  
THE POLYTHENE BAG SUPPLIED WITH THIS COVER.