

Sensory Screen

TO FIT the Excel Elise, Mobiquip XL, Budget XL & Excel Elise Neo special needs pushchairs

IMPORTANT: This is NOT a Sun Shade it DOES NOT protect from UVA/UVB sunlight

FITTING INSTRUCTIONS

Note. The Sensory Screen can only be used when a Sun Hood is fitted to the pushchair

START AT THE TOP - PLACE THE BROAD ELASTIC LOOPS OVER EACH HANDLE, LAY THE SCREEN OVER THE SUN HOOD AND DOWN IN FRONT OF THE SEAT.

NEXT THE SIDES - MOUNT THE MIDDLE EYELETS ONTO THE FRAME PINS AT THE SIDE OF THE SEAT (as you would when fitting a Rain Cover).

LASTLY THE BASE - MOUNT THE LOWER EYELETS ONTO THE FRAME PINS BY THE FOOTPLATE.

NOTE WHEN FOLDING THE PUSHCHAIR you will need to remove the Sensory Screen and Sun Hood to fold the pushchair.

CARE INSTRUCTIONS

When not in use store in the drawstring bag provided to keep in good condition.

Made from Stain Resistant Polyester fabric, hand wash only, no bleach, do not tumble dry or iron.

SAFETY ADVICE

The Sensory Screen is NOT a Sun Shade it DOES NOT protect from UVA/UVB sunlight, the carer is responsible for ensuring that the child is adequately protected with clothing and skin barrier creams from potentially harmful exposure to sunlight.

It is also necessary to check periodically that there is not a build up of heat inside the enclosed seat.

FOR THE SAFETY OF YOUR CHILD, PLEASE DESTROY
THE POLYTHENE BAG SUPPLIED WITH THIS COVER.